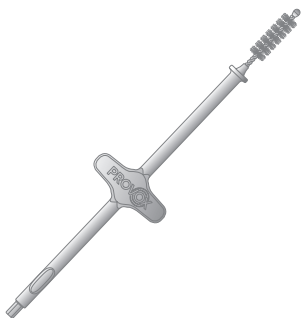
PROVOX[®] Brush

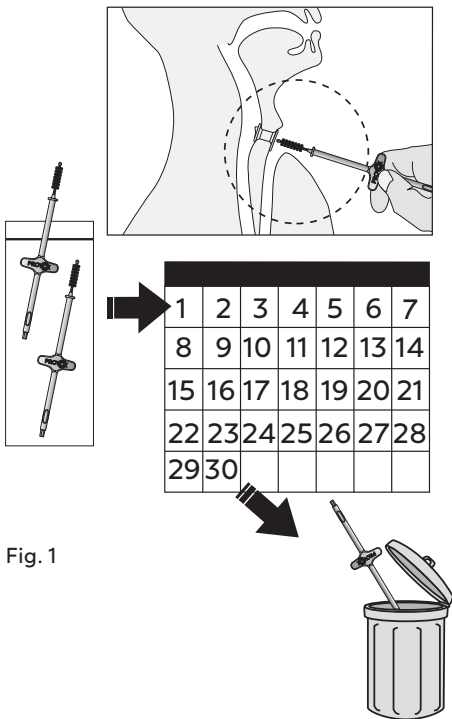
Fig. 1

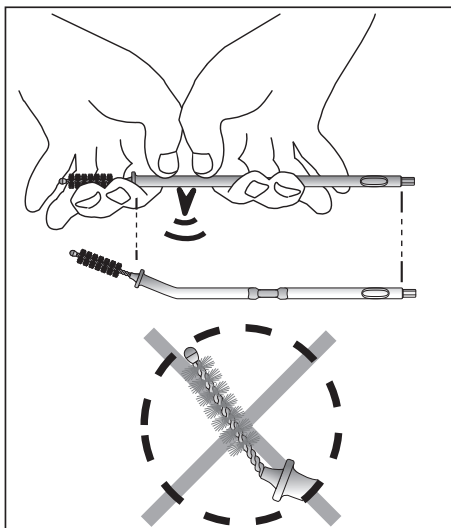
Fig. 2

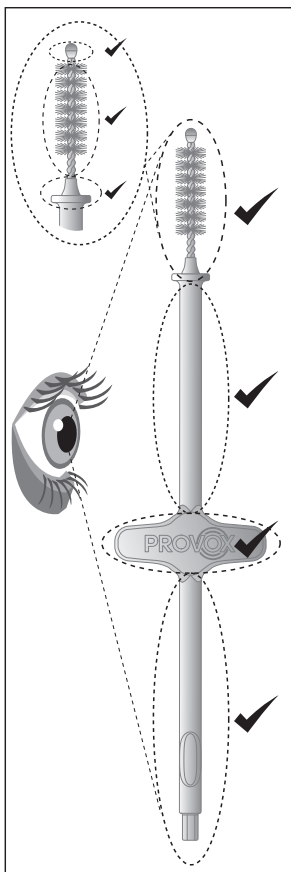
Fig. 3

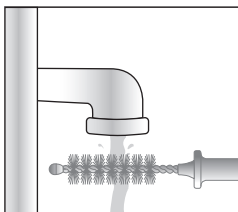
Fig. 4

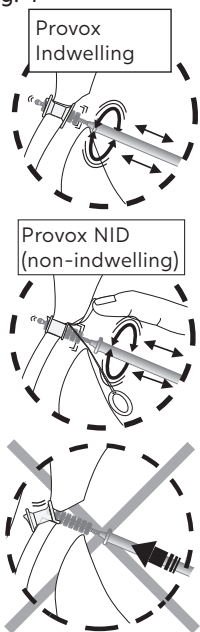
Fig. 5

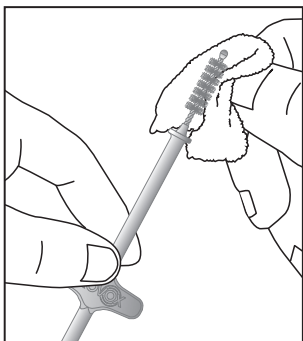
Fig. 6

2  / 250 ml (8,5 fl oz)

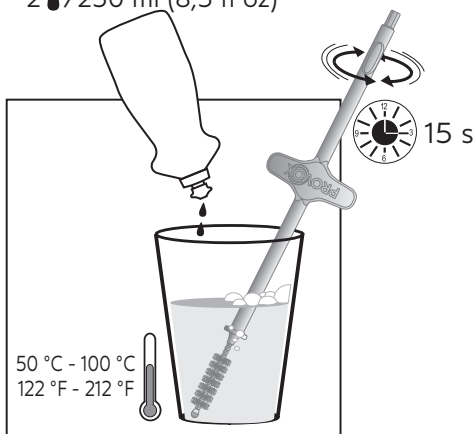
Fig. 7

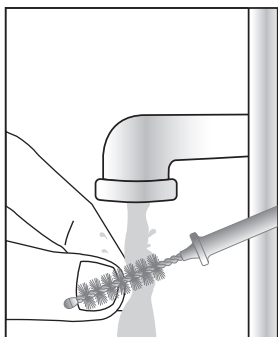
Fig. 8

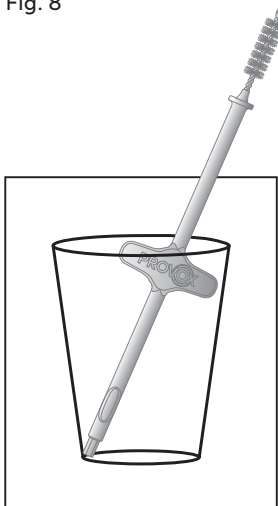
Fig. 9

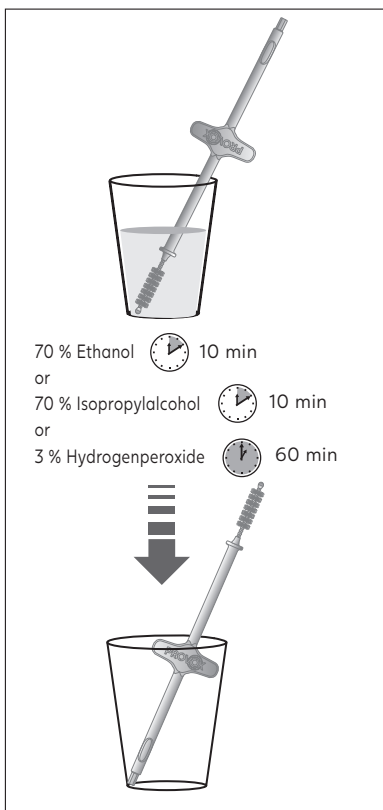
Fig. 10

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Compatible Products; Kompatible Produkte; Compatibele producten; Produits compatibles ; Suderinami gaminiai Productos compatibles Produtos compatíveis; Kompatibla produkter; Kompatible produkter; Kompatible produkter; Yhteensopivat tuotteet; Samhæfar vörur; Samhæfar vörur; Suderinami gaminiai; Kompatibilni produkty; Kompatibilis termékék; Kompatibilné produkty; Združljivi izdelki; Zgodne produkty; Produe compatibile; Kompatibilni proizvodi; Kompatibilni proizvodi; Συμβατά προϊόντα; Uyumlu Ürünler; Թացկցձաձոն ՅրոձըյԵԵձձոն; Համատեղելի արտադրանքներ; Uyğun Məhsullar; Совместимые изделия; Produk-Produk yang Kompatibel; Produk yang Serasi; 互換性のある製品; 호환 가능한 제품; 相容產品; מוצרים תואמים; מלנתج القلنتافوقة

Provox2 Voice Prosthesis
Provox Vega
Provox Vega XtraSeal
Provox Vega Puncture Set
Provox NID
Provox ActiValve
Provox ActiValve Lubricant
Provox LaryTube Fenestrated
Provox Life LaryTube Fenestrated
Provox Plug/Provox Vega Plug

CONTENTS

EN - ENGLISH.....	11
DE - DEUTSCH.....	12
NL - NEDERLANDS.....	13
FR - FRANÇAIS.....	14
IT - ITALIANO	15
ES - ESPAÑOL.....	16
PT - PORTUGUÊS.....	17
SV - SVENSKA	18
DA - DANSK.....	19
NO - NORSK.....	20
FI - SUOMI.....	21
IS - ÍSLENSKA	22
ET - EESTI.....	23
LV - LATVIEŠU	24
LT - LIETUVIŲ K.....	25
CS - ČESKY	26
HU - MAGYAR	27
SK - SLOVENČINA.....	28
SL - SLOVENŠČINA	29
PL - POLSKI.....	30
RO - ROMÂNĂ	31
HR - HRVATSKI.....	32
SR - SRPSKI	33
EL - ΕΛΛΗΝΙΚΑ.....	34
TR - TÜRKÇE.....	35
KA - ქართული.....	36
HY - ՀԱՅԵՐԵՆ.....	37
AZ - AZƏRBAYCAN DİLİ.....	38
RU - РУССКИЙ.....	39
ID - BAHASA INDONESIA.....	40
MS - BAHASA MELAYU	41
JA - 日本語.....	42
KO - 한국어.....	43
ZHTW - 繁體中文.....	44
HE - עברית.....	45
AR - العربية.....	46
ORDERING INFORMATION.....	47

Intended Use

Provox Brush is a single patient use device intended for cleaning Provox Voice Prosthesis, for insertion of Provox Plug and Provox Vega Plug, for applying lubricant and anti-candida agents and for cleaning of fenestration holes on Provox LaryTubes. The product is intended for use by the patient.

Before use

Always inspect the Provox Brush (fig.3) to make sure that it is not broken or worn out.

Before each use, make sure that

- the protective tip is not cracked or loose.
- the bristles do not appear worn or are loose.
- the wires are not bent or broken.
- the entire handle is not cracked or broken and that the safety wings have not been removed.

Provox Brush may be used for up to 30 days (fig. 1). If the brush shows any signs of damage, it must be replaced earlier.

Cleaning and disinfection

Clean the Provox Brush after each use according to fig. 6-9 using drinking water.

Disinfect the Provox Brush (fig. 10) with one of the following methods:

- Ethanol 70% for 10 minutes
- Isopropylalcohol 70% for 10 minutes
- Hydrogenperoxide 3% for 60 minutes

Do not use the device until it is completely dry

Note: Always clean and disinfect before use.

Precaution

- The device is intended for single patient use only. Reuse in another patient may cause cross contamination.
- Do not clean or disinfect by any other method as this might cause product damage and injury.
- Do not bend the brush head wire this can damage the product and cause injury, it is only allowed to bend the blue shaft (fig. 2).

Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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© Coloplast A/S. All rights reserved.
Atos and the Atos Medical logo are trademarks of Coloplast A/S.
Atos Medical AB