TRACOE Comfort

Contents

EN - ENGLISH.....	3
DE - DEUTSCH.....	5
NL - NEDERLANDS.....	7
FR - FRANÇAIS.....	9
IT - ITALIANO.....	11
ES - ESPAÑOL.....	13
PT - PORTUGUÊS.....	15
SV - SVENSKA.....	17
DA - DANSK.....	19
NO - NORSK.....	21
FI - SUOMI.....	23
LV - LATVIEŠU.....	25
LT - LIETUVIŲ KALBA.....	27
CS - ČEŠTINA.....	29
HU - MAGYAR.....	31
SL - SLOVENŠČINA.....	33
PL - POLSKI.....	35
RO - ROMÂNĂ.....	37
HR - HRVATSKI.....	39
SR - SRPSKI.....	41
EL - ΕΛΛΗΝΙΚΆ.....	43
TR - TÜRKÇE.....	45
KO - 한국어.....	47
ZHCN - 简体中文.....	49
HE - עברית.....	52
AR - العربية.....	54

Instructions for Use for Tracoe Paraffin Oil

REF 902

These instructions are for physicians, nursing staff and patients. They only apply to Tracoe Paraffin Oil.

Caution: Tracoe Paraffin Oil is only intended for use with Tracoe Comfort tracheostomy tubes.

1. Contents

The package contains 25 ml of pharmaceutical-quality liquid paraffin (DAB, Ph Eur, BP, NF, USP), supplied in a pharmaceutical dropper bottle.

It prolongs the life of the Tracoe Comfort tubes by permitting the inner cannula to slide in the outer cannula more easily.

2. Application

Tracoe Paraffin Oil is applied in order to permit the inner cannula to slide more easily in the outer cannula of Tracoe Comfort tracheostomy tubes.

After the tracheostomy tube has been cleaned and dried, a small amount (approx. 1 drop) of the Tracoe Paraffin Oil is applied to the outside of the inner cannula and distributed with a fluff-free cloth until a thin covering has been achieved. Then the inner cannula is inserted into the outer cannula and pulled back out and pushed back in several times in order to further distribute the liquid paraffin.

Caution: Make sure that the applied amount of the liquid paraffin is as small as possible. Never apply the oil to an extent that the liquid paraffin escapes between the inner and outer cannula and enters the trachea. There is a possibility, albeit small, that permanent overdosing may result in oil pneumonia.

Please do not replace the Tracoe Paraffin Oil by alternative products because other oils, for example stoma oil, might glue the inner cannula to the outer cannula.

3. Storage

The liquid paraffin is chemically inert and will not impair the Tracoe Comfort tracheostomy tubes in any way. The products must be stored in a cool and dry place. Temporary storage for ≤ 30 days at maximum $+50$ °C is possible. Protect the product from sunlight and extreme cold.

4. Returns and Complaints

If you have a complaint about the device, please contact complaint.se@atosmedical.com.

If it is involved in a reportable incident, as defined in local medical device legislation, additionally contact the appropriate regulatory body in the country of use.

5. General Terms and Conditions

The sale, delivery and return of all Tracoe products shall be affected exclusively on the basis of the valid General Terms and Conditions (GTC), which are available either from Tracoe Medical GmbH or on our website at www.tracoe.com.

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© Coloplast A/S. All rights reserved.
Atos and the Atos Medical logo are trademarks of Coloplast A/S.