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Instructions for Use for TRACOE humid assist Heat & Moisture Exchanger

REF 641 humid assist III

with paper filter, port for 15 mm connector
and optional oxygen supply port

REF 642 humid assist IV

with foam filter, port for 15 mm connector
and oxygen supply port

1. General Information

IMPORTANT: Please read these instructions for use carefully. They are an integral part of the described product and must be available at all times. The instructions refer exclusively to the above-mentioned products. Individuals using the product must be familiar with these instructions.

Storage: Store in a cool and dry place and protect from sunlight and extreme cold.

Contents of Pack: Pack of REF 641 contains 50 units, while the pack for REF 642 contains 30 units.

All TRACOE humid assist products are packaged individually in a clean room.

PLEASE NOTE: All TRACOE humid assist products are for use by one patient only and must not be cleaned or re-used. Dispose of the products in accordance with the valid national regulations for waste materials.

2. General Description

The TRACOE humid assist products are heat & moisture exchangers for single use. They ensure efficient humidification and warming of inspired air for tracheostomy patients, who are breathing spontaneously.

REF 641 and REF 642 have a female 15 mm connector that can be fitted to the standardised 15 mm connectors of tracheostomy tubes or endotracheal tubes.

3. Special Features

3.1 TRACOE humid assist III (REF 641)

This product is supplied with an oxygen supply port that can be inserted if required on the underside of the heat & moisture exchanger, after removing the occlusion cap.

3.2 TRACOE humid assist IV (REF 642)

The pressure relief valve on the side opposite the female 15 mm connector allows the expectoration of larger quantities of secretions.

4. Indications

All TRACOE humid assist products are heat & moisture exchangers that are suitable for patients in hospital or at home, who have a tracheostoma, but are capable of spontaneous breathing. They allow efficient humidification of the airways and warming of the inspired air. Their use reduces drying of the trachea and bronchi, and consequently a thickening of secretions, crust formation and the risk of atelectasis (alveolar collapse), and at the same time filters the inspired air.

5. Contraindications

TRACOE humid assist products must not be used in patients with heavy production of mucus in the airways, very viscous respiratory secretion or haemoptysis (coughing of blood) or in unconscious patients. They should also not be used in combination with external humidifiers or nebulisers.

TRACOE humid assist heat & moisture exchangers should be used within the recommended tidal volume range (see performance table on page 57 of the appendix), in order to prevent carbon dioxide accumulation, if the tidal volume is low, or insufficient moisturisation, if the tidal volume is too high.

6. Use

If the occlusion cap or oxygen connection / port is missing from the unopened packaging or if the occlusion cap is lying loosely in the packaging, the product must be disposed of and replaced by another product.

REF 641 and REF 642 are fitted to the 15 mm connector of a tracheostomy tube or endotracheal tube. A firm connection must be ensured. If an additional oxygen supply is necessary, the oxygen supply port should be connected to an oxygen source. For REF 641, the oxygen supply port is an optional extra and may be fitted as required to the underside of the heat & moisture exchanger, after removing the occlusion cap.

The maximum period of use for the heat & moisture exchanger is 24 hours. It should be replaced more frequently, if necessary.

7. Warnings

1. All TRACOE humid assist products are for single use only and must not be cleaned, disinfected, sterilised or reused, as this would permanently impair their function.
2. If a heat & moisture exchanger element becomes displaced within the housing, for example, as a result of coughing or during transportation, the product must be removed and replaced with a new one. On no account should any attempt be made to replace the displaced element.
3. Should the patient show heavy production of secretion or severe bleeding, this may result in blocking of the heat & moisture exchanger element. If there are signs of blockage, the heat & moisture exchanger should be removed immediately and replaced with a new one. On no account should any attempt be made to remove the blockage or clean the product. This can damage the heat & moisture exchanger element and lead to a considerable increase in respiratory resistance.
4. If TRACOE humid assist products are used with products from other suppliers, the fit must be carefully checked to exclude any possible risks. If used with adhesive carriers, it must be ensured that the HME cannot slip through the port of the adhesive carrier being used and fall into the trachea, as there is otherwise a risk of suffocation.

8. Returns

Returns of used products can only be accepted following prior consultation with the company and if accompanied by a completed decontamination certificate. You can obtain the required form either directly from TRACOE medical or from the website www.tracoe.com.

9. General Terms and Conditions

Sales, deliveries and returns of all TRACOE products are carried out exclusively according to the General Terms and Conditions, which can be obtained directly from TRACOE medical GmbH or from the website www.tracoe.com.

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